

### ▶ **Adjustment and service**

As with any quality product, proper care is needed to keep your loupes in peak condition. With normal daily use, your loupes will need service checks from time to time. If your loupe feels loose or does not fit as well as it did when new, contact your ExamVision dealer who will be able to help you with service and replacement parts. Your local eyecare professional may also be able to assist you with small adjustments or tightening of screws.

### ▶ **Correct use and maintenance**

Do NOT modify or alter your loupes. Inspect regularly for damage, and if you notice wear or damage to any components, or notice any loose parts, discontinue use and contact your ExamVision dealer immediately for service.

### ▶ **Allergies**

ExamVision uses only high-quality, allergy-tested eyeglass materials. Your loupes must be fitted so only the nose pad and temple tips touch your skin. In the highly unlikely event of any irritation, contact your ExamVision dealer.

### ▶ **Care and cleaning**

Do NOT use alcohol for cleaning. Use only ExamVision recommended alcohol-free cleaning and disinfectant products. Do NOT autoclave or use ultra-sonic cleaning methods.